

MADE IN THE



ADJUSTABLE FACE SHIELD



ATTENTION

PRODUCT INTENDED
FOR ONE TIME USE

see care instructions on back

P/N: SSP-LBT.FS

CLEAN SHIELD BEFORE USE

Shield intended for one time use by manufacturer

1. Wear disposable gloves
2. Wash with mild soap and water to clean. Dry.
3. Remove gloves and follow process after each use.

WEAR INSTRUCTIONS

1.) Adjust strap to your head size with the sliding buckle. Slide buckle to the right to tighten, and slide buckle to the left to loosen.



2.) Pull adjustable elastic over and around the crest of your head. Adjust to your preferred comfort level.